


## **Non-Contact Infrared Themometer**

Contact: Annie

Emai: annie@aliance-china.com.cn

+86 21 6252 5669 Mobile: +86 135 8576 5009 Skype: Annieding1986

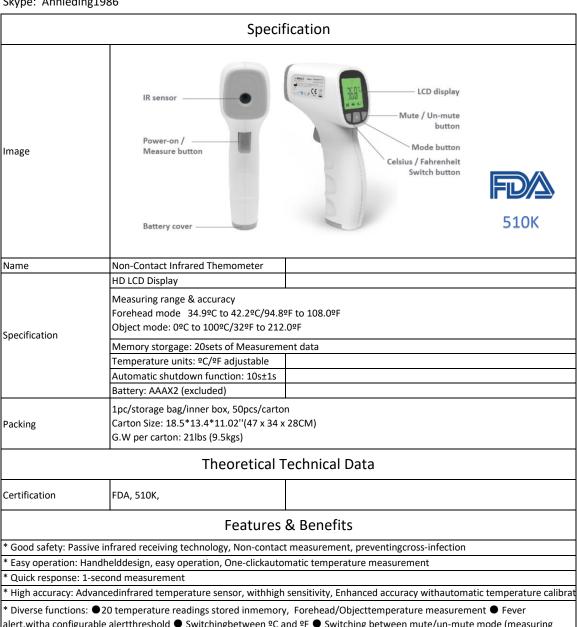

alert,witha configurable alertthreshold ● Switchingbetween ºC and ºF ● Switching between mute/un-mute mode (measuring sound notification) ● Automatic power-off, power-saving

## Precautions, Warnings & Tips

- The thermometer issuitable for anindoorenvironment without strong airconvection between the thermometerand the target.Forexample, windsfrom a fan, an air-conditioner, or aheater.
- \* Do not holdthe thermometer for a long time, because itissensitive to the ambient temperature.
- \* Makesure the sense head isfree of foreign mattersbefore use;
- \* Makesure the forehead hasno sweat and no hairscovered before measure the forehead temperature; otherwise, the result couldbe incorrect;
- No intense emotion or strenuous exercisesbefore measuring